

Monofocal IOLs

Designed to deliver without compromise

MADE IN UK



RayOne
ASPHERIC

RayOne
SPHERIC

 Rayner

RayOne fully preloaded IOL injection system, designed to deliver without compromise

The need

Talking to surgeons indicates that they are looking for an injector that can deliver an IOL consistently, with expert control, through a micro incision with minimal wound stretch.

The dilemma? The preloaded IOL systems available to date have sought to meet these requirements by enhancing one element, be it the injector or lens. However, this means that a trade-off has to be made, usually between the ease of use or surgical outcomes.

At Rayner, we believe that the only way to create a true, fully preloaded micro incision cataract surgery (MICS) injection system that works consistently without compromise, is to design the system as one - both lens and injector. This was the inspiration behind RayOne.

The solution

When creating RayOne, we developed our MICS lens and unique patented Lock & Roll technology as part of the same design process; this combination has resulted in the smallest fully preloaded injector available (1.65 mm nozzle).

Our RayOne MICS lens is an enhanced version of the tried-and-tested C-flex platform, now in a 6 mm optic design.

We have retained the material and design benefits of our original lens, without compromising on proven stability or optical performance.

RayPRO

Real-time patient feedback data

RayPRO is a mobile and web-based digital platform that collects insightful Patient Reported Outcomes (PROs) over three years.

• New insightful trends

- Promote your services to new patients, with easy-to-understand metrics.
- Supports appraisals, recertification and auditing.

• Fast and simple

- Patients are registered in seconds.
- Only value-adding data is collected.
- Access metrics anytime from your smartphone (iOS and Android).

• Automated collection of PROs

- Patients provide their feedback in just a couple of minutes.
- Responses are anonymous to encourage patient honesty.
- Reports are always live, with no data analysis needed.
- Use product and patient trends to improve your service.

• Secure cloud-based platform

- Designed for data security and to be GDPR and HIPAA compliant.
- Only you can see your personal RayPRO reports and metrics.
- Questionnaire responses are non-identifiable to protect patients' data privacy.

Comparison of preloaded IOLs

Comparison of Preloaded IOLs			
Company	Rayner	Alcon	Johnson & Johnson
Lens platform	600C / 100C	Acrysof IQ	Tecnis1
Injector	RayOne	UltraSert	iTec
Nd:YAG rate	1.7% ¹	7.47% ⁷	3.75% ⁷
Miyata grade (glistenings)	0 ² (None)	3 ⁸ (High)	0 ¹² (None)
Abbe value	56 ²	37 ⁹	55 ⁹
Refractive index	1.46 ³	1.55 ¹⁰	1.47 ¹²
Mean decentration	0.08 mm ⁴	0.78 mm ¹¹	0.27 mm ¹³
Nozzle diameter	1.65 mm ⁵	2.08 mm ⁵	1.86 mm ⁵
Injector preparation steps	2 ⁶	3 ¹⁰	4 ¹²

References:

1. Mathew RG and Coombes AGA. Ophthalmic Surg Lasers Imaging. 2010 Nov-Dec; 41(6):651-5. 2. Rayner. Data on File. White paper. 3. Ferreira T et al. J of Refract Surg. 2019; 35(7): 418-25 4. Bhogal-Bhamra GK et al. Journal of Refractive Surgery. 2019;35(1):48-53. 5. Nanavaty MA et al. J Cataract Refract Surg. 2009; 35:663-671. 6. www.rayner.com 7. Cullin F et al. Acta Ophthalmol. 2014; 92(2): 179-83 8. Werner L. J of Refract Surg. 2010; 36(8): 1398-1420 9. Zhao H et al. Br J Ophthalmol. 2007; 91(9): 1225-29 10. www.mylcon.com 11. Humbert G et al. FR J Ophthalmol. 2013; 36(4): 352-61 12. jnvisionpro.com 13. Baumeister M et al. J of Refract Surg. 2009; 35(6): 1006-12



RayPRO is FREE for users of Rayner IOLs.
rayner.com/raypro



RayOne with patented Lock & Roll technology for a smoother, more consistent rolling and delivery of the lens via micro incision

RayOne enhanced 6 mm optic

Available as a spheric or aberration-neutral aspheric lens



Amon-Apple enhanced square edge for minimal PCO 1.7% at 24 months⁷

Zero glistenings⁸

A large fully preloaded power range is available

- One solution for all your patients
- Aspheric and spheric: +8.0 to 34.0 D


Based on proven haptic technology for excellent stability^{1,2}

Reliable optical outcomes and a low rate of post-operative complications

Rayner's anti-vaulting haptic technology provides excellent fixation in the capsular bag¹:

- **Superb centration** - Average offset of only 0.08 mm 3 to 6 months after surgery²
- **Excellent rotational and torsional stability** - 1.83° mean IOL rotation 3 to 6 months after surgery²

Stability of RayOne IOLs

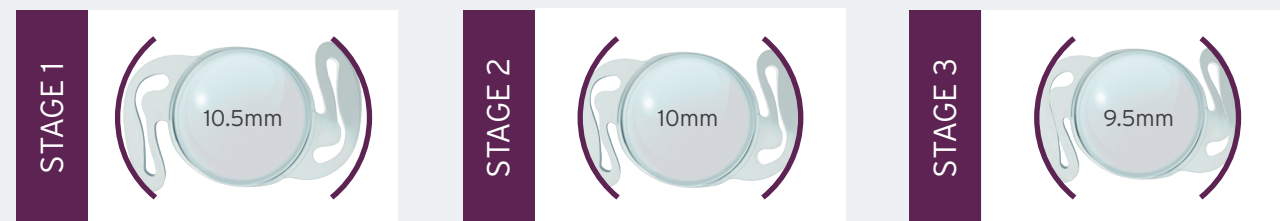

Outer haptics begin to take up the compression forces of post-operative capsule contraction

Outer haptics engage the inner haptics


Haptic tips gently meet the IOL optic and are effectively locked into position

RayOne easy to use injector

Sub 2.2 mm incision

1.65 mm RayOne nozzle for sub 2.2 mm incision

- Smallest fully preloaded injector nozzle
 - Ease of insertion
 - Enables true micro incision
- Parallel sided for minimal stretch
 - Sub 2.2 mm delivery
 - Maintains incision architecture

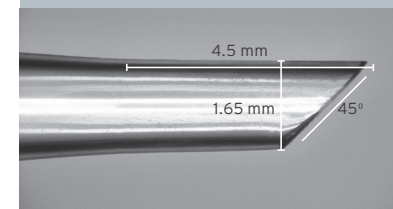


Ergonomic design for ease of handling

True 2-step system


- Simple and intuitive
 - Minimal learning curve
 - Minimizes error
- Increase efficiencies
 - Designed for repeatability
 - Reduces operating time
- **Step 1:** Insert OVD into cartridge via port
- **Step 2:** Lock cartridge ready for implantation

Single handed plunger with minimal force required

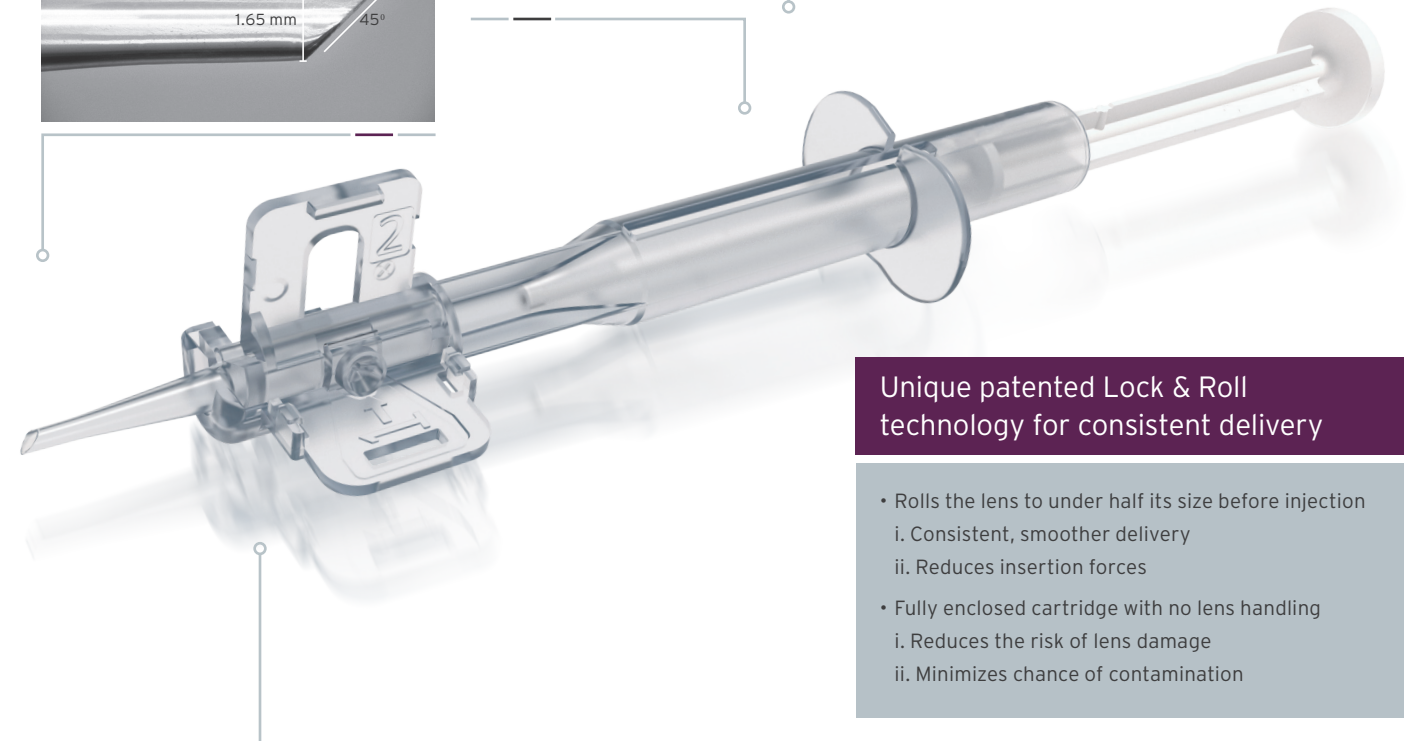

Unique patented Lock & Roll technology for consistent delivery

- Rolls the lens to under half its size before injection
 - Consistent, smoother delivery
 - Reduces insertion forces
- Fully enclosed cartridge with no lens handling
 - Reduces the risk of lens damage
 - Minimizes chance of contamination

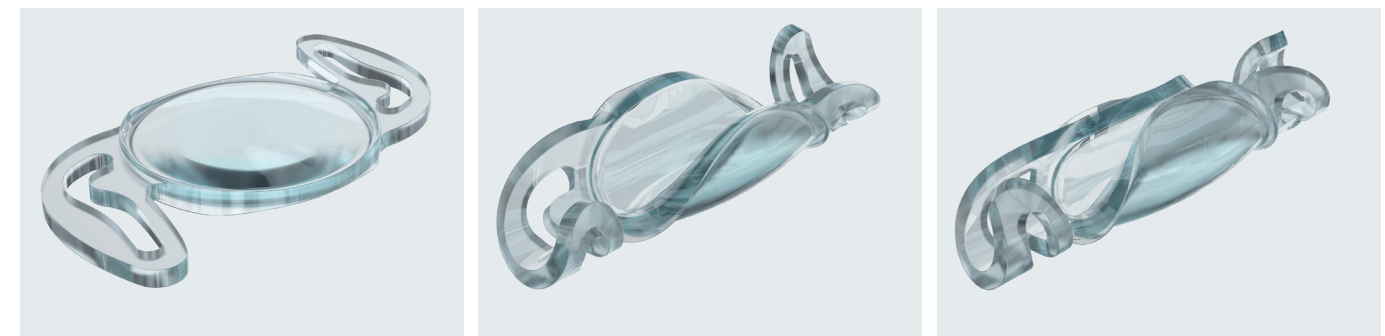

References: 1. Clauoué C. Clinical and Surgical Ophthalmology 2008; 26(6):198-200. 2. Bhogal-Bhamra GK et al. Journal of Refractive Surgery. 2019;35(1):48-53. 3. Nanavaty MA et al. J Cataract Refract Surg. 2009; 35:663-671. 4. Yagci R et al. Eur J Ophthalmol. 2014 Jul 24; 24(5):688-92. 5. Vyas AV et al. J Cataract Refract Surg 2007; 33:81-87. 6. Johansson B et al. J Cataract Refract Surg. 2007; 33:1565-1572. 7. Mathew RG and Coombes AGA. Ophthalmic Surg Lasers Imaging. 2010 Nov-Dec; 41(6):651-5. 8. Rayner. Data on File. White paper. 9. McLoone E et al. Br J Ophthalmol. 2001; 85:543-545. 10. Richter-Mueksch S et al. J Cataract Refract Surg. 2007; 33:1414-1418. 11. Altmann GE et al. J Cataract Refract Surg. 2005; 31(3): 574-585. 12. Vyas, A. Journal of Clinical and Experimental Ophthalmology. 2015; 6:391. 13. Tomlins PJ et al. J Cataract Refract Surg. 2014; 40:618-625. 14. PePOSE JS et al. Graefes Arch Clin Exp Ophthalmol. 2009 Jul;247(7):965-73. 15. Nanavaty MA and Kubrak-Kisza M. J Cataract Refract Surg. 2017 Apr;43(4):558-563. *Of those who expressed a preference.

When considering an intraocular lens, what's important to you?

RayOne Aspheric offers aberration-neutral technology for optimal visual quality and functional visual acuity in all light conditions

RayOne Aspheric is designed with an aspheric anterior surface that creates no spherical aberration.

Studies have demonstrated that aberration-neutral technology:

- Offers improved contrast sensitivity compared with spherical IOLs^{3,4}
- Provides better low light level visual acuity than spherical IOLs⁴
- Can offer more depth of field than aberration-negative IOLs by retention of the patient's natural level of corneal spherical aberration⁶
- Are less susceptible to the effects of decentration than aberration-negative IOLs¹¹
- Twice as many patients* preferred the aberration-neutral IOL than aberration-negative⁶
- Three times fewer reports of visual disturbances with the aberration-neutral IOL than aberration-negative⁶

Reducing dysphotopsia by design¹²

- Rayner's Enhanced Square Edge Technology shows no general increase in glare from previous models without a square edge⁷
- Low refractive index (1.46)

360° Optimised Barrier to reduce PCO

Rayner's 360° Amon-Apple Enhanced Square Edge creates an optimum barrier to reduce epithelial cell migration including at the haptic-optic junction^{5,7}. Extremely low Nd:YAG capsulotomy rates, comparable with hydrophobic acrylic lenses with square-edge optics.⁷

ND: YAG CAPSULOTOMY RATES ⁷		MEAN TIME TO ND: YAG CAPSULOTOMY ⁷
At 12 months	0.6%	9.3 ± 5.5 mths (range 2.6 - 22.7 mths)
At 24 months	1.7%	Follow-up period: 5.3 - 29 mths


Also available as RayOne Spheric, a spherical monofocal with many of the same optic platform benefits, fully preloaded in the RayOne injector.

Vacuole free material for a glistening free IOL⁸

- Single piece IOL created from a homogeneous material free of microvacuoles⁸
- Compressible material for delivery through a micro incision
- Excellent handling characteristics with controlled unfolding within the capsular bag
- Low silicone oil adherence⁹
- Excellent uveal biocompatibility¹³


RayOne preloaded primary IOL platform not only meets your surgical needs, it exceeds them

- **Lock & Roll technology**
 - Rolls the lens to under half its size before injection for consistent and smooth delivery
 - Fully enclosed cartridge with no lens handling means low risk of lens damage and contamination
- **Easy to use, two step system**
 - Simple design minimizes the learning curve and chance of errors
 - Efficient system reduces your operating time
- **1.65 mm nozzle**
 - Smallest fully preloaded injector nozzle available
 - Sub 2.2 mm delivery for low risk of SIA
- **A large fully preloaded power range is available**
 - One system for all your patients
 - From +8.0 D to +34.0 D

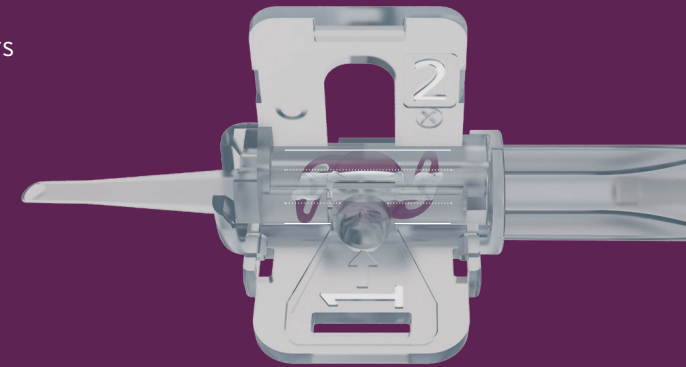


In a comparative study of six market-leading preloaded delivery systems¹⁵

1. RayOne received the maximum score for 'ease of use' for all delivery steps:

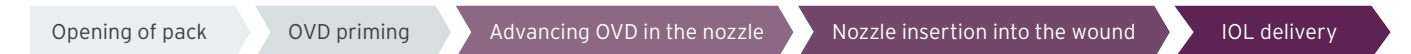

2. RayOne was the least time consuming system for delivering the IOL
3. RayOne showed less injector tip damage post-insertion than 50% of the tested delivery systems
4. RayOne showed minimal wound stretch compared to other tested delivery systems when through a 2.2 mm incision

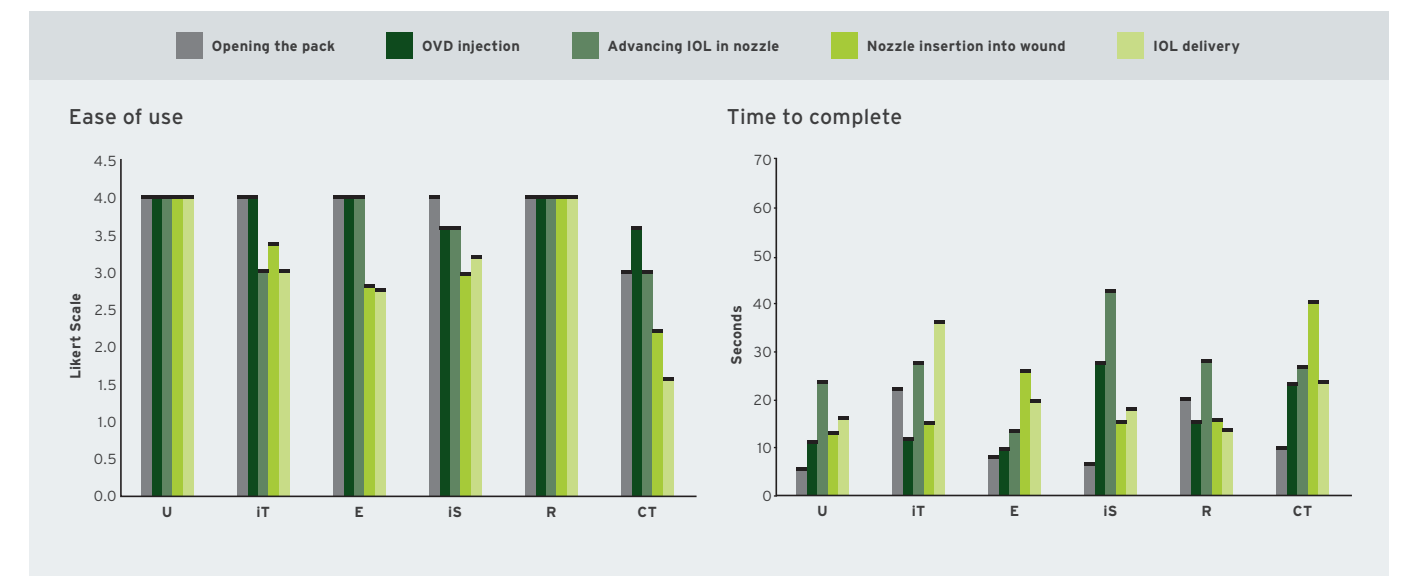

Ultrasert (U) (Alcon Laboratories, Inc.), iTec (IT) (Abbott Medical Optics, Inc.), Eyecee (E) (Bausch & Lomb, Inc.), iSert (IS) (Hoya Surgical Optics, Inc.), RayOne (R) (Rayner Intraocular Lenses Ltd.), and CT Lucia (CT) (Carl Zeiss Meditec AG). All trademarks are property of their respective owners

RayOne Aspheric and Spheric Intraocular Lenses (IOLs)

The IOL preloaded within the Rayner RayOne injection system is a Rayner 600C aspheric and 100C spheric. The Rayner 600C aspheric and 100C spheric has optical and material equivalence to the Rayner C-flex 570C and 970C IOLs.

The combined IOL and injection system is referred to as RayOne Aspheric (RAO600C) and RayOne Spheric (RAO100C).

INDICATIONS: The Rayner 600C and 100C intraocular lenses are indicated for primary implantation for the visual correction of aphakia in adults in whom a cataractous lens has been removed by phacoemulsification. The lens is intended to be placed in the capsular bag.

CONTRAINDICATIONS: Apart from non-specific contraindications related to any form of ocular surgery, the following specific contraindications must be respected.

1. Microphthalmia
2. Active ocular disease (e.g. chronic severe uveitis, proliferative diabetic retinopathy, chronic glaucoma not responsive to medication)
3. Children under the age of 21 years
4. Corneal decompensation or corneal endothelial cell insufficiency
5. Persons who are pregnant or nursing

WARNINGS: A risk/benefit ratio must be assessed before confirming a patient as a candidate for the Rayner 600C or 100C IOL implantation, if they are suffering from any of the following conditions:

1. Recurrent ocular disease (e.g. uveitis, diabetic retinopathy, glaucoma, corneal decompensation)
2. Previous ocular surgery

3. Non-age-related cataract
4. Vitreous loss
5. Iris atrophy
6. Severe Aniseikonia
7. Ocular Hemorrhage
8. Macular degeneration
9. Zonular dehiscence
10. Ruptured posterior capsule
11. Patients in whom the intraocular lens may affect the ability to observe, diagnose, or treat posterior segment diseases.
12. Surgical difficulties at the time of cataract extraction which might increase the potential for complications (e.g. persistent bleeding, significant iris damage, uncontrolled positive pressure, or significant vitreous prolapse or loss)
13. A distorted eye due to previous trauma or developmental defect in which appropriate support of the IOL is not possible.
14. Circumstances that would result in damage to the endothelium during implantation.
15. Suspected microbial infection.
16. Children under the age of 2 years are not suitable candidates for intraocular lenses.

Since the Rayner 570C C-flex® IOL clinical study was conducted with lens implantations into the capsular bag only, there is insufficient data to demonstrate the safety and efficacy for ciliary sulcus placement of the 570C, 600C, or 100C IOLs.

ATTENTION: Reference the Instructions for Use labeling for complete listing of indications and precautions.

CAUTION: Federal U.S law restricts this device to the sale by, or on the order of a physician.

RAYNER USA OFFICE

Rayner Surgical Inc.
1460 Broadway
New York, NY 10036

RAYNER USA CUSTOMER SERVICE & DISTRIBUTION CENTER

Metro Park Warehouses, Inc.
6901 Stillwell, Kansas City MO 64120
Tel: 866-961-1811
Fax: 866-956-5029

RAYNER GLOBAL HEADQUARTERS

Rayner Surgical Group Limited
10 Dominion Way, Worthing
West Sussex, BN14 8AQ. United Kingdom



Discover why RayOne is in a class of its own visit rayner.com/USA

Regulatory restrictions may apply in certain markets.

©2022 Rayner Surgical Inc. Unauthorized reproduction prohibited.

'Rayner', the RayOne logo and 'Lock & Roll' are proprietary marks of Rayner. EC 2021-46 US 03/22

RayOne Aspheric and Spheric Technical Information

Model Name:	RayOne Aspheric and Spheric
Model Number:	RayOne Aspheric (RAO600C)
Model Number:	RayOne Spheric (RAO100C)
Power Range:	+8.0 to +29.5 D (0.5 D increments) +30.0 to +34.0 D (1.0 D increments)

Delivery System	
Injector Type:	Single use, fully preloaded IOL injection system
Incision Size:	1.65 mm nozzle for sub 2.2 mm incision
Bevel Angle:	45°
Lens Delivery:	Single handed plunger

Monofocal IOL	
Material:	Single piece Rayacryl hydrophilic acrylic
Water Content:	26% in equilibrium
UV Protection:	Benzophenone UV absorbing agent
UV Light Transmission:	UV 10% cut-off is 380 nm
Refractive Index:	1.46
ABBE:	56
Overall Diameter:	12.50 mm
Optic Diameter:	6.00 mm
Optic Shape:	Biconvex
Asphericity:	Anterior aspheric surface with aberration-neutral technology (RayOne Aspheric)
Optic Edge Design:	Amon-Apple 360° enhanced square edge
Haptic Angulation:	0°, uniplanar
Haptic Style:	Closed loop with anti-vaulting haptic (AVH) technology

Estimated Constants for Optical Biometry								
SRK/T	Haigis			HofferQ	Holladay	Holladay II	Barrett Universal II	
A-constant	a0	a1	a2	pACD	SF	pACD	LF	DF
118.6	1.17	0.40	0.10	5.32	1.56	5.32	1.67	0

For Contact Ultrasound, the estimated A-constant for Aspheric is 118.0.

Please note that the constants indicated for all Rayner lenses are estimates and are for guidance purposes only. Surgeons must always expect to personalize their own constants based on initial patient outcomes, with further personalization as the number of eyes increases.




Discover why RayOne is in a class of its own visit rayner.com/USA

Regulatory restrictions may apply in certain markets.


©2022 Rayner Surgical Inc. Unauthorized reproduction prohibited.

'Rayner', the RayOne logo and 'Lock & Roll' are proprietary marks of Rayner. EC 2021-46 US 03/22


RayOne fully preloaded IOL injection system

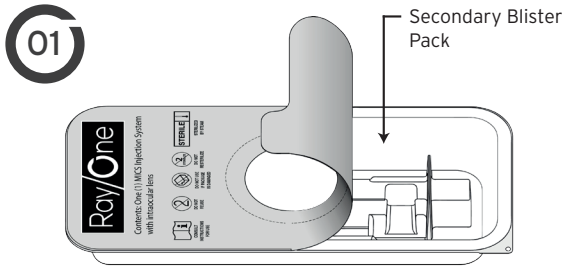


Completely peel back the lid of the first blister pack.

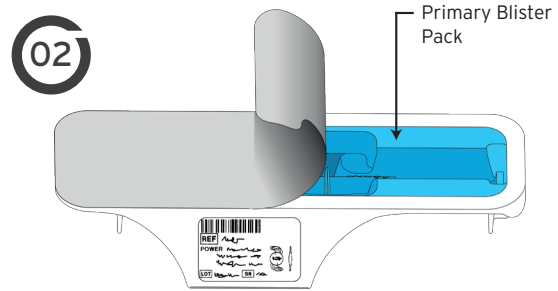


Carefully peel back the lid half way down the second blister tray.

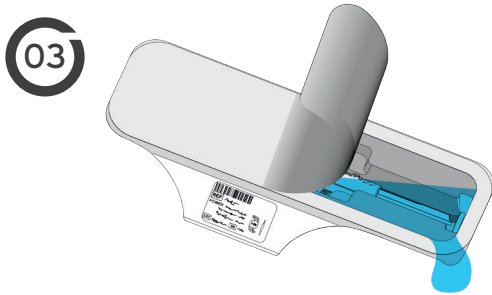


Carefully drain the saline from the blister tray and peel off the remaining lid. **Do not remove the injector from the blister tray.** Do not wait more than 3 minutes before adding OVD - due to the risk of dehydration.

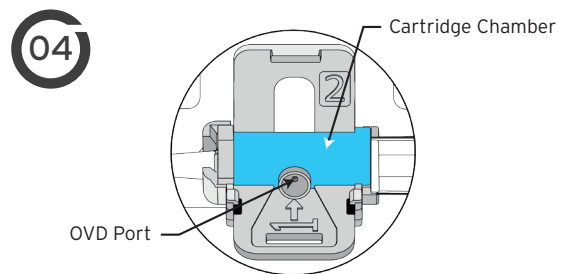


Insert the OVD cannula into the opening marked with an arrow on the cartridge and apply sufficient OVD to completely fill the cartridge.

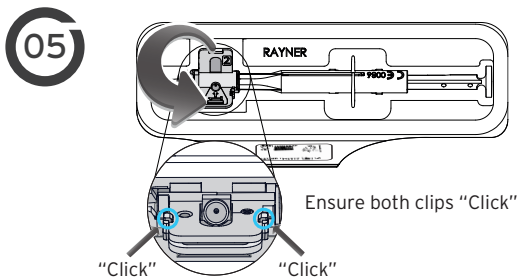


Keep the injector in the tray and close the cartridge firmly together by pushing the moving half of the cartridge (labelled 2) against the fixed half until you hear it click closed. **Check both clips have "clicked" shut and secured the cartridge.**

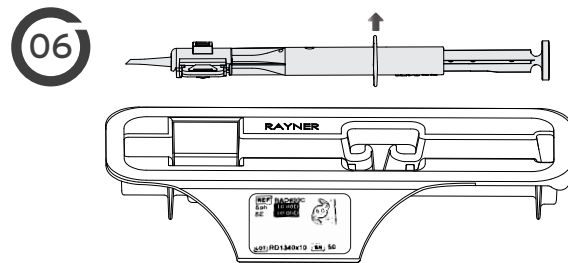

Gently lift out the injector from the tray.

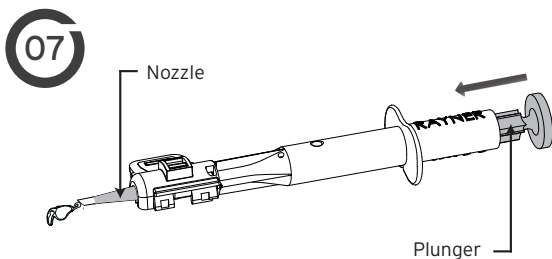

Press the plunger in a slow and controlled manner. If excessive resistance is felt this could indicate a blockage, discontinue use of the product and return the product and all packaging to Rayner. In the case of IOL rotation during ejection from the nozzle, gently rotate the injector in the opposite direction to counteract any movement.

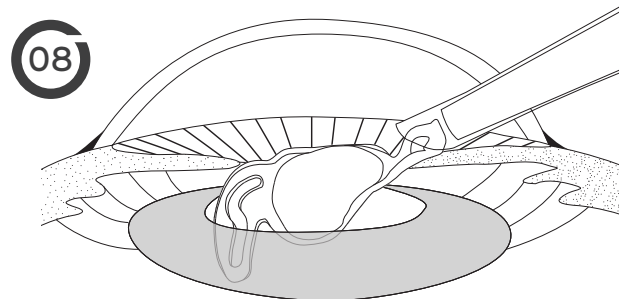

Stop depressing the plunger when the IOL exits the nozzle. Discard the injector after use. Following implantation, irrigate/aspirate to eliminate any OVD residues from the eye, especially behind the IOL.